

INSTRUCTIONS FOR USE



Operatory Protection

Polycoated Headrest Covers

Instructions for use:

1. Use a surface disinfectant as recommended by chair manufacturer to clean the headrest.
2. Hold cover with opening on the bottom.
3. Place cover over headrest until headrest is completely covered.



Ref.#'s L0BL, L0DR, L0GR, L0LV,
L3GR, L3LV, LOWH, L3WH, L3BL,
L3DR, L13BL, L13WH, L15WH,



Do not reuse.



Not made with natural rubber latex.



CROSSTEX[®]
A CANTEL MEDICAL COMPANY

EC REP MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Made in USA

Manufactured by: Crosstex International, Inc.
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

POLYHEADCVR-IFU
Rev. 0 - 5/17