

Clinically Proven
Adhesive Cement System

Multilink® Automix

Universal Adhesive Cement



A strong
bond,
proven performance.



**ivoclar
vivadent!**
passion vision innovation

CLEAN



Clean Restoration

- Rinse restoration with water, then air dry.
- Cover entire bonding surface with Ivoclean.
- Allow to react for 20 seconds then rinse and air dry.

PRIME



Prime Restoration

- Apply Monobond Plus to the bonding surface of the restoration.
- Allow to react for 60 seconds. Air dry.

BOND



Bond Restoration

- Bond with Multilink Automix

Simplified Bonding



Step 1 - Prep Dentin/Enamel

- Scrub 1:1 mixture of self-etching primer into tooth structure for 30 seconds.
- Air dry.

Step 2 - Cement

- Dispense Multilink Automix into restoration and seat.

Step 3 - Clean-Up

- After seating, light-cure each quarter surface for 1-2 seconds. The cement will achieve a gel-like consistency for easy clean-up.
- Light cure buccal and lingual margins for 20 seconds each.

Step 4 - Post-op

- Final post-op view

DELIVERY FORMS

Multilink® Automix System Packs

Contents:
 1 – Multilink Automix Syringe (9g)
 1 – Multilink A/B Primer (2 x 3g)
 1 – Monobond Plus (5g)
 1 – Liquid Strip (2.5g)
 Various Accessories



Shades:
 627471WW Transparent
 627473WW Yellow
 627472WW Opaque
 645954WW White

Multilink® Automix Refills

Contents:
 1 – Automix syringe (9g)
 15 – Mixing tips



Shades:
 615216WW Transparent
 615217WW Yellow
 615218WW Opaque
 645952WW White

Multilink® Automix Try-in Pastes

645956WW Transparent
 645957WW Yellow
 645958WW Opaque
 645959WW White

637568AN Ivoclean
 626221AN Monobond Plus (5g)
 576825WW Multilink Primer
 A+B Refill (2x3g)

613626WW Multilink Primer Refill A (3g)
 613627WW Multilink Primer Refill B (3g)
 532505AN Liquid Strip (2.5g)

Accessories

592435 Multilink Applicator Reg & Endo (x50)
 645951 Mixing Tips short tapered Refill / 15
 645955 Mixing Tips- Root Canal / 5

United States
 175 Pineview Drive
 Amherst, NY 14228
 716.691.0010
 800.533.6825
 Fax: 716.691.2285

Canada
 1 - 6600 Dixie Road
 Mississauga, Ontario L5T 2Y2
 905.670.8499
 800.263.8182
 Fax: 905.670.3102

©2013 Ivoclar Vivadent, Inc.
 Multilink, Monobond Plus, Ivoclean are trademarks of Ivoclar Vivadent.

633248 3/2013

ivoclarvivadent.com

