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Bosworth Original Truliner Instructions for Use

Original Truliner is a PMMA denture corrective relining material used to restore a denture's original fit.

DIRECTIONS

1. Clean and dry denture. Roughen denture surface (2mm) and periphery for best adhesion.
2. Coat buccal and labial surfaces with Bosworth Masque.
3. Slowly add 1 full plastic vial measure of powder (15ml) to 1 full glass vial measure of liquid (7.6ml) in mixing cup. Spatulate for 30 seconds until material is a honey-like consistency.
4. Apply mixture to all surfaces to be bonded. When material is flat in color and has absorbed all liquid, rinse patient's mouth and insert denture.
5. Have patient close to align occlusion.
6. Keep denture in patient's mouth until material is in a dough-like stage (approx. 1 minute).
7. Remove from patient's mouth before final set. Rinse mouth with cold water. Adjust as needed and polish.

NOTES

1. Before making a permanent denture relin, check for sores or cancer and treat the tissue with a tissue conditioner such as Bosworth Softone to eliminate the possibility of the final relin duplicating the disturbed or deformed mucosa caused by the deficient denture.
2. Recommended: Bosworth's Pressure Pot for a smoother, color-stable and reduced porous relin.
3. Do not allow denture to final cure in the mouth.

CAUTION: Certain individuals may be sensitive to products containing acrylic.

MIX RATIO	SHELF LIFE
7.6ml (1 full glass vial) liquid to 15ml (1 full plastic vial) powder	3 years