OptiBond[®]
All-In-One

Instruction For Use

Bonding Agent

OptiBond® All-In-One Self-Etching Dental Adhesive

Description

OptiBond All-In-One self-etching adhesive combines the ingredients needed for etching, priming, and bonding into a single adhesive solution, thereby eliminating separate etching and priming steps of the bonding process. OptiBond All-In-One adhesive can be used for the bonding of both direct and indirect restorations.

Indications for use

Direct Applications

Light-cured composite and compomer restorations.

Composite/ceramic/ metal repairs.

Cavity sealing for amalgam restorations.

Light-cured or dual-cured core build-ups (always light-cure an incremental build-up of core material.).

Indirect Applications

Veneers.

Porcelain, composite, and metal-based inlays, onlays, crowns, bridges.

Endodontic posts.

Cavity sealing as a pretreatment for indirect restorations.

Important: For indirect applications, use NX3 resin cement whenever there is no or limited light accessibility. NX3, either self-cured or dual-cured, is compatible with OptiBond All-In-One adhesive.

OptiBond All-In-One adhesive can be used to prime metal, ceramic, and composite substrates/restorations.

Contraindications

1. Patients who have a history of severe allergic reaction to methacrylate resins.
2. Direct pulp capping.
3. In combination with a chemical-cure or dual-cure material when there is insufficient light-curing. **NX3 resin cement is an exception.**

Instructions for Use

Direct Restorations

Bonding Composites/Compomers to Dentin/Enamel

1. Prepare cavity. Wash thoroughly with water spray and air dry (Do not desiccate).
Note: For optimal adhesion, always prep (bevel) all enamel and/or margins prior to application of OptiBond All-In-One adhesive.
2. Shake OptiBond All-In-One adhesive bottle briefly.
Note: Upon first use, shake OptiBond All-In-One adhesive bottle vigorously for 10 seconds.
3. **Bottle Delivery:** Dispense 2-3 drops of OptiBond All-In-One adhesive into a clean well. Replace cap immediately after dispensing.
Unidose™ Delivery – for single patient use only: Open the unidose device and insert the applicator brush into the container to saturate the tip.
4. Using the disposable applicator brush, apply a generous amount of OptiBond All-In-One adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds.
5. Apply a second application of OptiBond All-In-One adhesive with a brushing motion for 20 seconds.
6. Dry the adhesive with gentle air first and then medium air for at least 5 seconds with oil-free air.
7. Light cure for 10 seconds*.
8. Apply composite according to manufacturer's instructions for use.

Indirect Restorations -- See indication-specific information in sections A-D below.

Veneers, crowns, bridges, inlays, onlays and metal-based restorations (used in conjunction with a luting cement)

1. Clean the preparation. Wash thoroughly with water spray and air dry (Do not desiccate).
2. Shake OptiBond All-In-One adhesive bottle briefly.
Note: Upon first use, shake OptiBond All-In-One adhesive bottle vigorously for 10 seconds.

- Bottle Delivery:** Dispense 2-3 drops of OptiBond All-In-One adhesive into a clean well. Replace cap immediately after dispensing.
Unidose Delivery – for single patient use only: Open the unidose device and insert the applicator brush into the container to saturate the tip.
- Using the disposable applicator brush, apply a generous amount of OptiBond All-In-One adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds.
- Apply a second application of OptiBond All-In-One adhesive with a brushing motion for 20 seconds.
- Dry the adhesive with gentle air first and then medium air for at least 5 seconds with oil-free air. Avoid pooling of adhesive before light curing – refer to sections B & C.
- Light cure for 10 seconds*.

Indication Specifics for Indirect Restorations

A. Veneers - Apply luting cement to veneer, seat the veneer, and cure the cement according to manufacturer's instructions for use.

B. Crowns, bridges, inlays, onlays and metal-based restorations – Avoid pooling of adhesive before light curing. Following light curing of the adhesive, apply luting cement (see important note below for cement recommendation) to the restoration or prep, seat the restoration, and cure the cement according to manufacturer's instructions for use.

Important Note: Use only NX3 resin cement whenever there is no or limited light accessibility (e.g. metal based, zirconia-based, and alumina-based substrates). When there is good light-accessibility, use any Kerr dual-cure permanent cement including NX3.

C. Post Bonding - Apply adhesive on post and prep. Avoid pooling of the adhesive before light-curing. Remove excess adhesive with a dry applicator brush or absorbent paper point. Following light curing of the adhesive, apply NX3 dual-cure resin cement onto the post, seat the post, vibrate the post slightly avoiding the possibility of trapped air then cure NX3 according to the manufacturer's instructions.

D. Core Bonding - Apply core material according to manufacturer's instructions for use.

Note: Use a light-cured or dual-cured core material. Always light-cure an incremental build-up of core material.

Refrigerate upon receipt between 2° – 8°C.

***Recommended Cure Times when using Kerr Demetron**

Lights:

Optilux 501 10 seconds

L.E.Demetron I 10 seconds

L.E.Demetron II 5 seconds

Demi 5 seconds

For all other lights see manufacturer's recommendation.

CAUTION

1. **OptiBond All-In-One adhesive bottle should be tightly closed immediately after use.**
2. Corrosive Liquid: Avoid contact with skin, eyes and soft tissue. In case of contact with skin or eyes, flush immediately and thoroughly with water. Get medical attention for eyes. Do not take internally.
3. Uncured methacrylate resin may cause contact dermatitis.
4. OptiBond All-In-One adhesive is a light-cured material. Once the material is placed in a mixing well, use immediately or protect from ambient light.
5. OptiBond All-In-One adhesive contains acetone and is flammable. Keep away from sources of ignition.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact.

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Limited Warranty - Limitation of Kerr's Liability

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr Corporation
Kerr Australia Pty. Ltd.
Sybron Dental Specialties Japan, Inc.