

Coprwx Bite Wafers (84's)

Coprwx is specifically manufactured to help insure proper relation of models when mounting on an articulator. It is impregnated with copper powder to provide uniform heat transmission during the warming and cooling process.

Coprwx Bite Wafers contain a sheet of foil to insure micrometric clearance, strengthen the bite, prevent the teeth from cutting through, and reduce distortion.

Suggestions for use:

Soften Coprwx in warm water (110°-120° F.) before establishing the Occlusal relationship of the patient by having him close into centric relation. The wax may then be chilled with cold water (40°-50° F.) and used to mount the working models on the articulator.