



VibrakleenTM E²

Enzyme Ultrasonic Cleaner Effervescent Tablets

For general purpose ultrasonic cleaning,
evacuation system cleaning and pre-soak

Two Tablets Make One Gallon/3.785ℓ

Clinic Pack/80 Tablets

DIRECTIONS FOR USE: TWO TABLETS MAKE ONE GALLON. Add two tablets to one gallon of warm water. For two quarts or less, use one tablet. Do not mix with any other cleaning or disinfecting product. Wear eye protection and rubber gloves while handling.

FOR ULTRASONIC CLEANERS: Depending on size of ultrasonic unit, drop either one or two tablets directly into tank. Fill with warm tap water. Water temperature should not exceed 140°F (60°C). If pre-mixing for later use, place two tablets in a gallon container and fill with warm tap water. Store mixed solution in a covered container until ready to use. Ultrasonically clean items for 10 minutes. Some items may require longer, but extended time in solution can promote rusting on some types of steel or carbides. Place burs in a bur block to avoid trauma to bur heads. Rinse items thoroughly. Dry instruments before proceeding with the sterilization process. Replace solution when it becomes visibly cloudy, or minimally, once a day as recommended by the “ADA Recommendations for Dental Office Procedures.”

FOR EVACUATION SYSTEMS: At the end of each day, add two VibraKleen E² tablets to one gallon of tap water. Draw at least one quart of solution through each suction outlet and run system for a few minutes as recommended by the manufacturer. If heavy surgical procedures are performed, flush lines several times daily.

FOR INSTRUMENT PRE-SOAK: Soak instruments in solution for at least 10 minutes, longer for tougher stains. Do not exceed temperature of 140°F. Do not leave any instrument in the solution longer than 24 hours.

Typical pH of solution 7.5–8.5. Please refer to manufacturers directions of dental instruments used in solution.

For technical service information, please call 1-800-431-1785.

WARNING: AVOID INHALATION OF
VAPORS. DO NOT BRING IN CONTACT
WITH EYES OR SKIN. CONTACT MAY
CAUSE IRRITATION.

KEEP OUT OF REACH OF CHILDREN

NOT FOR INGESTION

KEEP IN A COOL, DRY PLACE

Patent No. 5529788

Reorder #50037101

Manufactured by:

Heraeus Kulzer, Inc.

99 Business Park Drive

Armonk, NY 10504 USA

1-800-431-1785

MADE IN U.S.A.

53710183 R11/03