

MARK3 Ultrasonic Enzyme Cleaner is a concentrated instrument pre-soak.

Ultra Sonic Cleaning Solution:

With advanced biomass changing dynamics, this ultrasonic cleaner will degrade a wide range of complex organics and soils that traditional protease enzymatic formulations cannot. MARK3 Ultrasonic Enzyme Cleaner Tablets are an anti-corrosive, anti-hard water mineralization formula that was designed for use on all your dental instruments that may become soiled with blood and other organic debris. (Removes blood, protein and tissue from dental instruments, metal trays, eye wear protection, equipment and clothing.)

New Anti-Corrosive Formula:

Prolong the life of your valuable instruments with our new formula. Studies have proven that using this new highly concentrated anti-corrosive formula will prolong the life of your instruments compared to other ultrasonic solutions.

Directions For Use:

Place one tablet for a typical 3 quart office unit, for large volume units that hold 4-6 quarts, use two tablets in an ultrasonic bath water.

For cleaning purposes, it is preferable to use warm tap water not to exceed 140 F degrees. Manually clean instruments with residual debris with a handled brush. Instruments should be sonicated for a minimum of 10 minutes. For tough stains add 5 or more minutes to the procedure. Rinse instruments with water to remove any excess product residue, dry and start your sterilization process.

Soaking instruments overnight as a daily procedure is not recommended: however if this procedure is preferred, the new anti-corrosive formula will continue to protect the instruments from any corrosion.

General Purpose Cleaner:

For light cleaning around the office, use two tablets per gallon of water.

Evacuation Cleaner:

Can be used as a secondary evacuation cleaner. Use 1 tablet per 1/2 gallon of warm water.

WARNING:

Wear gloves and eye protection when handling this product. In the event of contact with skin or eyes, wash immediately with water.
Keep out of reach of children. NOT FOR HUMAN CONSUMPTION.

MARK3™

Bioenzymatic Ultrasonic Cleaner

Made from natural products, biodegradable

USA/CANADA classified as biological safety level 1

Non foaming, non chlorine

Non corrosive, neutral ph 6.5 - 7.5

Fresh scent, eliminates malodors at their source



Eco-Friendly Product