Prior to use, carefully read the instructions for use.

EN

GC Dry Mouth Gel

TO PROVIDE EXTRA COMFORT FOR DRY MOUTH SUFFERERS

GC Dry Mouth Gel is designed to provide comfort for person suffering from Dry Mouth. It is recommended to be used for people who may experience difficulties in eating, speaking or are suffering from discomfort due to dry mouth. GC Dry Mouth Gel is sugar free.

RECOMMENDED INDICATIONS

- To provide comfort and relief for dry mouth conditions.
- To provide a protective coating for teeth and oral tissue.
- To help control bad breath.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to hydroxybenzoates.

DIRECTIONS FOR USE

- Use GC Dry Mouth Gel as often as required when the mouth feels dry. This can be anytime during the day or at night time.
- Extrude a sufficient amount of gel from the tube on the finger or use a cotton swab to take the gel from the tube. Apply the gel to tongue, oral mucosa and or teeth.

FLAVOUR

Fruit Salad (F)*, Raspberry (R), Orange (O), Lemon (L), Mint (M)
*Not available in Europe.

F, R, O, L and M in parentheses are abbreviations for each flavour.

These abbreviations are marked on the tubes after lot numbers to show the flavour of the contents.

For example, 060815MMint

STORAGE

Recommended for optimal performance, store in a cool dark low humidity area away from direct sunlight (8-25°C)(46.4-77.0°F).

PACKAGE

A tube contains 40g (35mL).

- 10 tube pack (2 each of 5 flavours)
- 10 tube pack (4-Flavour pack)
- 10 tube pack (Raspberry flavour)

Note:
All packaging types may not be available in all countries.


CAUTION

- Avoid contact with eyes. In case of contact, flush immediately with plenty of water and seek medical attention.
- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- If spilled onto clothing, wash with water.
- Anyone who is taking prescription medications should consult a doctor before using this product.
- GC Dry Mouth Gel can be used in combination with GC Tooth Mousse / GC MI Paste or GC Tooth Mousse Plus / GC MI Paste Plus but should not be used with other products.
- In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com
or for The Americas
http://www.gcamerica.com
They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 09/2018

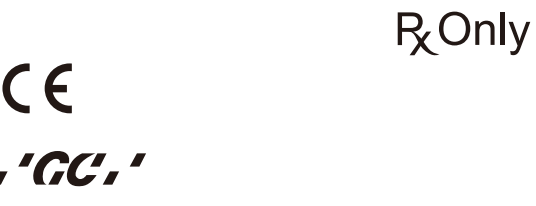MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

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