

Adper™ Single Bond Plus Adhesive

Etch:

- Apply Scotchbond™ Etchant to enamel and dentin. Wait 15 seconds.
- Rinse for 10 seconds.
- Blot excess water using a cotton pellet or mini-sponge. **Do not air dry.**
- The surface should appear glistening without pooling of water.



Adhesive:

- **Immediately after blotting**, apply 2-3 consecutive coats of adhesive for 15 seconds with gentle agitation using a fully saturated applicator.
- Gently air thin for 5 seconds to evaporate solvent.



Light-Cure:

- Light-cure for 10 seconds.



Restorative:

- Place restorative according to manufacturer specifications.

