

Adper™ Single Bond Plus Adhesive

Lightly clean the root surface with flour of pumice. Rinse and blot dry.

Etch:

- Apply Scotchbond™ Etchant to the dentin. Wait for 15 seconds.
- Rinse for 10 seconds.
- Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.



Adhesive:

- Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator.
- Gently air thin for 5 seconds to evaporate solvents.
- Light cure for 10 seconds.
- Apply 2 additional coats of adhesive.
- Dry gently for 5 seconds.
- Light cure for 10 seconds.



- Remove the oxygen inhibited layer with a moistened gauze.



1 of 1