Flexi-Flow CEM®
Recommended for prefabricated post systems.

TITANIUM REINFORCED COMPOSITE CEMENT

Flexi-Flow Cem® with Titanium, a composite cement with the compressive strength of dentin, was developed to enhance the retention of all prefabricated posts. The unique ingredients in the base and catalyst gel mix together to create a composite with the low viscosity you want, and the compressive strength and permanence that post placement requires.

UNIQUE FEATURES OF FLEXI-FLOW CEM WITH TITANIUM

- Compressive strength of 44,729 p.s.i. (comparable in strength to natural dentin).
- Chemically bonds to composite core build-up, eliminating time consuming removal of excess cement.
- Reduces hydrostatic pressure, and coats the post-hole and post effectively.
- Takes just *Five Minutes* to mix, place, set and harden.

TO PREVENT PROBLEMS WITH MIXING

- ALWAYS keep syringes tightly capped when not in use.
- ALWAYS dispose of plastic spatulas after using to prevent contamination.
- ALWAYS store at room temperature.
- NEVER return any gel to syringes after dispensing.

STEPS FOR MIXING

- Dispense catalyst past from the syringe onto the mixing pad.
- Dispense an equal amount of base paste onto the same mixing pad.
- With the wedge shaped end of the plastic spatula, mix the catalyst paste and the base paste together for about 30 seconds to achieve thorough blending. (Flexi-Flow Cem with Titanium gives you 3 minutes working time before it sets.)
- To make Flexi-Flow set faster, add more base. To make Flexi-Flow set slower, add more catalyst.**

MIX, PLACE, SET, HARDEN –ALL IN JUST FIVE MINUTES

After mixing Flexi-Flow Cem with Titanium for 30 seconds, place it onto the post and into the post-hole using a Lentulo spiral drill, or any instrument you feel appropriate. Insert post, let set undisturbed for the next 4 minutes.

The post is now ready for core build-up.

NOTES:

- To make use of the maximum tensile strength of Flexi-Flow when used for cementing posts, the canal should be etched or grooved (use Essential Dental Systems’ Grooving Drills, Cat. No. 593-0 or Cat. No. 593-123).**

- Flexi-Flow should not be used with any bonding agent in the canal. Bonding agents within the canal greatly accelerate the setting time of Flexi-Flow Cement.**

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